

TROUBLE-SHOOTING

IF YOUR B.07 STAIRLIFT WILL NOT OPERATE

- Check swivel seat. Is seat locked in riding position?
- Check footrest. Is there an obstruction on the stairway?
- Check residence main circuit breaker (or fuse) panel. Has a breaker tripped or a fuse blown?
- Make a visual check. Do you notice anything about your stairlift that is different or out of place?
- If the B.07 Stairlift does not operate after completing the above check-list, contact your area dealer to perform all required services, repairs and maintenance.

36 MONTH LIMITED WARRANTY

Ensure your Savaria Dealer provides you with a copy of the manufacturer's limited parts warranty and documentation relating to any Dealer labour warranty.

FOR OWNER'S RECORDS

Customer Name: _____

Installing Dealer: _____


Dealer's Telephone Number: _____

Date Installed: _____

Serial/Job Number: _____



B.07 STAIRLIFT

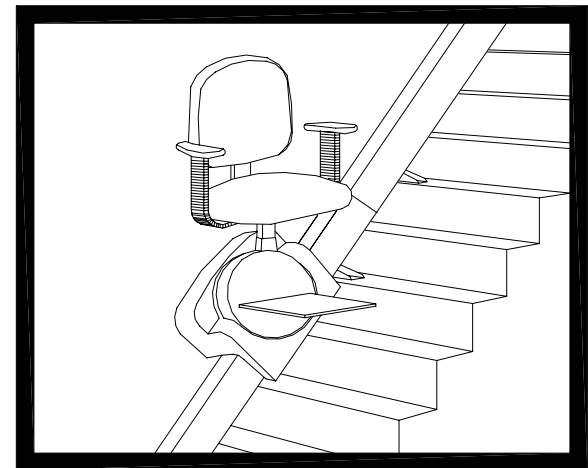

OWNER'S MANUAL

Part No. 000659
01-m04-2010

Thank you for purchasing a Savaria B.07 Stairlift.
We are sure your B.07 Stairlift will enhance your mobility.

Enjoy it.

SPECIFICATIONS

Applications	: Indoor Straight run stairway Outdoor (optional)
Maximum travel	: 32 ft (9754 mm)
Speed	: 18 fpm (0.09 m/s) - approx.
Angle of incline	: 25° to 45°
Rated load	: 300 lb (136 kg)
Drive system	: Roller chain/sprocket
Motor	: ½ hp, 110 VDC Runs on a standard household current 110 VAC
Rail	: Aluminum extrusion

MAINTENANCE

Every 3 months

These verifications can be made by the owner or a qualified technician.

Verification of safety switches

- Swivel seat switches
- Footrest sensor switches

These procedures are to verify that the stairlift does not run when those components are not in operating position.

Every 6 months

These verifications must be made by a qualified technician.

- Top and bottom final limit switches
- Slack chain switch
- Safety brake test

As required

- Lubrication of rollers (white grease)
- Lubrication of pulleys (refer to noise)
- Lubrication of the seat swivel system (wheel bearing grease)

Note: The maintenance is not covered by warranty.

OPERATING PROCEDURES

- 1- Position yourself in the seat and fasten the seat belt. Raise and hold the seat swivel handle, and then rotate the seat so that you are parallel to the rail (with your back against the stairway wall). Release the seat swivel handle to secure the seat in that position.

A SAFETY SWITCH PREVENTS THE LIFT FROM OPERATING UNLESS THE SEAT IS LATCHED IN POSITION (PARALLEL TO THE RAIL).

Position both feet securely on the footrest and place your arms on the armrest cushions.

- 2- The UP/DOWN rocker switch is located below the front end of the right armrest. Pressing and holding one end of the rocker control switch will cause the carriage to move in that direction. The carriage stops automatically at the upper or lower end of the rail or if the switch is released.
- 3- Two remote CALL/SEND control units enable you to "call" or "send" the carriage to/or from either end of the rail.

SAFEGUARDS

Be sure stairs are clear of objects before operating lift.
Fasten your seat belt before operating lift.
Seat must be locked in "riding" position for UP/DOWN travel.

SAFETY FEATURES

Your B.07 Stairlift is equipped with a safety brake that will prevent the carriage from going down in the event of a chain rupture.

Safety switches installed on the B.07 Stairlift stop the carriage when it reaches the top or bottom of the stairway.

The footrest incorporates obstruction sensors which will stop the carriage if an obstacle is encountered on the stairs. If this should occur, press the opposite side of the "UP/DOWN" control switch to run the carriage in the opposite direction. Clear the obstacle from the stairway and proceed as usual.

Your B.07 Stairlift will not run if the position of the seat is different than the normal traveling position. (90° degrees).

SEATING FEATURES

The seat assembly of the B.07 Stairlift is designed to swivel and lock in five positions.

Lifting the seat swivel handle allows you to swivel the seat to a convenient position for transferring into or out of the seat.

The seat and the footrest can easily be folded up to allow more space on the stairway.

Your B.07 Stairlift is equipped with flip-up armrests (to allow easier transfer from wheelchair or scooter). To raise an armrest, lift the armrest and flip it backwards.

When seated, lower the armrest back to its original position. Push it down and ensure it is locked.

B.07 Stairlift

